

Fall Fighter™ Sensor Pad Monitors

Thank you for your purchase of this Fall Fighter™ Sensor Pad Monitor. Crest Healthcare Supply warrants this product to be free from defects in material and workmanship for a period of one (1) year from the date of shipment of the products to you.

WARNING: *The effectiveness of Fall Fighter™ Monitors and Sensor Pad Systems depends on proper equipment installation and operation. Always test this system prior to use. Fall Fighter™ Sensor Pad Systems are designed to help alert staff of situations that can lead to or result in falls, and are not a substitute for proper caregiver supervision. Please ensure that patients at high risk for falls are sufficiently monitored and help limit environmental hazards that can add to fall risk. For best results, only Fall Fighter™ Monitors and Sensor Pads should be used together.*

Instructions for Fall Fighter™ Bed or Chair Monitor Set-Up:

- 1) Remove the screw from the battery compartment door on the back of the monitor using a Phillips head screwdriver.

NOTE: *This screw is optional and is not required to hold the battery compartment door in place.*

- 2) Insert 3 AAA batteries into the monitor in the correct orientation as shown inside the monitor.

NOTE: *For safety, ensure that the batteries have sufficient power for proper monitoring. When battery is low, the front LED will flash blue every 5 seconds and an audible chirp will sound.*

- 3) Adjust the tone, volume and delay settings on the monitor to fit the user by using the appropriate controls inside the battery compartment.

NOTE: *If using the "Mute" option on the Bed Monitor, the monitor must be connected to the nurse call system using a cable for alarm annunciation. No local alarm will sound.*

- 4) Close battery compartment door. Secure with screw if desired.

- 5) Attach the metal mounting bracket to the back of the monitor using the included screws and a Phillips head screwdriver.

- 6) Connect sensor pad to monitor by plugging modular plug end into the appropriate jack on the monitor.

NOTE: *Verify that sensor pad is not expired prior to use.*

- 7) Place the sensor pad on the desired monitoring surface (bed or chair) and attach monitor securely to a nearby area (such as a wheelchair back or headboard) using the included metal clip.

NOTE: *Sensor Pads require sufficient pressure to compress pad when the user is positioned on it. If using over a cushion or other soft surface, test system before use to make sure the pad can be properly compressed.*

- 8) Place the user on the pad surface to activate system and test to ensure function. (If placed properly, the monitor will sound an alarm to notify staff if the user removes weight from the pad.)

For Questions or Troubleshooting Assistance, Contact:

CREST

HEALTHCARE SUPPLY®

800-328-8908

www.cresthealthcare.com

PO Box 727

195 Third Street South
Dassel, MN 55325-0727